

# *Medi-Fridge*™

## Micro-Cooler

Owners Use and Care Manual

Model MF-MC-3V1P-401-B

Patent pending.

The American Diabetes Association states: “Good insulin care begins with how the insulin is stored. Insulin does not work well...when exposed to extreme temperatures. It will stay fresh...as long as its temperature stays under 86°F.” (Copyright © 2006 American Diabetes Association, From <http://www.diabetes.org>, Reprinted with permission from The American Diabetes Association). In other words, temperatures over 85°F may damage or destroy your medicines’ effectiveness or potency.

**OVERVIEW:** Your *Medi-Fridge* Micro-Cooler was designed to provide a convenient and affordable way to keep medicines (such as insulin or other medicines that are injected) at a safe temperature for up to 16 hrs., when you do not have access to a power source while traveling, at work, at home, in vehicles, even outdoors:

**STORAGE:** Your *Medi-Fridge* Micro-Cooler will hold up to three 10ml vials ( 7/8” diameter and 2-1/8” length) or one pen (up to 1” diameter and 7” in length)

**USE and CARE:** Your *Medi-Fridge* Micro-Cooler includes three main components:

- **One (1) insulated nylon bag with a zipper closure.**
  - o This can be cleaned with a damp cloth or towel, baby wipes work very well. Do not place this or any **Micro-Cooler** components in a power washing machine.
  - o The top of the bag has a small pouch that will hold other supplies such as syringes, test strips, and swabs.
- **Two (2) reusable ice packs.**
  - o Use only one at a time. The second ice pack should be kept in a freezer (no colder than 0°F) so it is ready when you need it.
  - o The reusable ice packs can be cleaned with mild soap and warm, not hot, water. Do not place this or any **Micro-Cooler** components in a power washing machine.
  - o To prepare the reusable ice pack for use, place it in a freezer with the side marked “FREEZE WITH THIS SIDE UP” up in a freezer for 7 hours or more. The freezer temperature should be no lower than 0°F. Extended exposure to temperatures below 0°F will damage the ice pack and void the warranty.
  - o Prior to placing the frozen ice pack in the **Micro-Cooler** bag, allow the ice pack to stand in room temperature for ten (10) minutes or place the frozen ice pack in a microwave at high power for one (1) minute.
  - o Place the reusable ice pack in the cavity at the bottom of the **Micro-Cooler** bag with the side marked “PLACE IN THE MICRO-COOLER BAG WITH THIS SIDE UP” facing upward.
- **One (1) Medicine Cradle Foam Insert.**
  - o The foam insert can be cleaned with a damp cloth or towel, baby wipes work very well.

### **INSRCTIONS FOR USE:**

- o Place the frozen, reusable ice pack in the bottom of the **Micro-Cooler** bag (with the writing facing upwards). Place the Foam Insert in the bag on top of the reusable ice pack.
- o The cavity in foam insert will accommodate up to 3 10ml vials of medicine or one pen type medicine dispenser.
- o Gently place the medicine in the cavity until they touch the reusable ice pack below.
- o Close the top of the **Micro-Cooler** bag and seal with the zipper.
- o Keep the **Micro-Cooler** bag away from extreme heat sources and direct sunlight to ensure that your medicine to stay at a safe temperature longer.

# ***Medi-Fridge***<sup>SM</sup> Micro-Cooler

## LIMITED WARRANTY FOR PRODUCTS AND ACCESSORIES PURCHASED IN THE UNITED STATES OR CANADA

Subject to the exclusions contained below, ***Medi-Fridge*** warrants its **Micro-Cooler** to be free from defects in materials and workmanship under normal consumer usage for the period of one (1) year. This limited warranty is a consumer's exclusive remedy, and applies as follows to new ***Medi-Fridge*** products and accessories purchased by consumers in the United States or Canada, which are accompanied by this written warranty:

### EXCLUSIONS

**Normal Wear and Tear.** Periodic maintenance, repair and replacement of parts due to normal wear and tear are excluded from coverage.

**Abuse & Misuse.** Defects or damage that result from: (a) improper operation, storage, misuse or abuse, accident or neglect, such as physical damage (cracks, scratches, etc.) to the surface of the product resulting from misuse; (b) contact with harsh chemicals or submersion in water or exposure to rain, sand, dirt or the like, extreme heat, or food; (c) use of the products or accessories for commercial purposes or subjecting the product or accessory to abnormal usage or conditions; or (d) other acts which are not the fault of ***Medi-Fridge***, are excluded from coverage.

**Unauthorized Service or Modification.** Defects or damages resulting from service, testing, adjustment, installation, maintenance, alteration, or modification in any way by someone other than ***Medi-Fridge***, or its authorized service centers, are excluded from coverage.

**Altered Products.** Products or Accessories that show evidence of tampering.

***Medi-Fridge***, at its option, will at no charge repair, replace or refund the purchase price of any products or accessories that does not conform to this warranty. We may use functionally equivalent reconditioned/refurbished/pre-owned or new products or accessories or parts.

To obtain service or information, please call: (512) 255-2271. You will receive instructions on how to ship the products or accessories, at your expense, to a ***Medi-Fridge*** authorized repair center. To obtain service, you must include: (a) a copy of your receipt, bill of sale or other comparable proof of purchase; (b) a written description of the problem, and, most importantly; (e) your address and telephone number. This warranty extends only to the first consumer purchaser, and is not transferable.

ANY IMPLIED WARRANTIES, INCLUDING WITHOUT LIMITATION THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL BE LIMITED TO THE DURATION OF THIS LIMITED WARRANTY, OTHERWISE THE REPAIR, REPLACEMENT, OR REFUND AS PROVIDED UNDER THIS EXPRESS LIMITED WARRANTY IS THE EXCLUSIVE REMEDY OF THE CONSUMER, AND IS PROVIDED IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. IN NO EVENT SHALL ***Medi-Fridge*** BE LIABLE, WHETHER IN CONTRACT OR TORT (INCLUDING NEGLIGENCE) FOR DAMAGES IN EXCESS OF THE PURCHASE PRICE OF THE PRODUCT OR ACCESSORY, OR FOR ANY INDIRECT, INCIDENTAL, SPECIAL OR CONSEQUENTIAL DAMAGES OF ANY KIND, OR FOR FINANCIAL LOSS ARISING OUT OF OR IN CONNECTION WITH THE ABILITY OR INABILITY TO USE THE PRODUCT OR ACCESSORIES TO THE FULL EXTENT THESE DAMAGES MAY BE DISCLAIMED BY LAW.

Some states and jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitation on the length of an implied warranty, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights that vary from state to state or from one jurisdiction to another.

**Copyright:** ***Medi-Fridge***<sup>TM</sup> MICRO-COOLER, the ***Medi-Fridge*** logo, and any other trademarks or service marks are owned by Technological Investments, LLC and ***Medi-Fridge***. No ***Medi-Fridge*** trademark may be used without prior written permission. Any unauthorized use of the trademark or service mark is prohibited. No licenses or right to use are granted by implication estoppels or otherwise in respect of any copyrights, patents, trademarks, services marks in relation to this product.